

# SOL-M™ Blunt Fill Needle

with and without filter



SOL-M Blunt Fill Needles help to reduce the risk of needlestick injuries when drawing-up and preparing medications. Blunt Fill Needles eliminate the unnecessary use of sharps.

## Easy Identification - Colour coded hub.

- Blunt Fill Needle (BFN) for use with vials.
- Blunt Fill Needle with 5 micron filter (BFNF) for use with glass ampoules.

## Ease of Use

- Both needles have a conventional needle hub compatible with all standard syringes.

## Reduction in Medication Contamination

- The 5 micron filter in the hub of the BFNF means any glass particles larger than 5 microns are filtered out preventing them from being drawn into the syringe prior to injection.





# SOL-M™ Blunt Fill Needle

with and without filter

Medication Preparation

## Usage Guidelines

Remove the Blunt Fill Needle from its package and connect to the syringe.

Remove the needle cap.

BFN - Insert the needle into the centre of the vial stopper at a 90° angle. This will help reduce the risk of rubber fragments from the side of the vial stopper contaminating the medication.

BFNF - Draw up medication from glass ampoule.

Once medication has been prepared and is in the syringe ready for administration, remove the BFN/BFNF and dispose of in a sharps container.

The Blunt Fill Needle is for **single use only**. Reuse will dull the needle, remove the silicone coating and increase the chance of coring and rubber fragmentation.



18 Gauge  
45° Bevel



- Red – Blunt Fill Needle
- Purple – Blunt Fill Needle with Filter

Reference number	Product description	Gauge size	Outer diameter	Length	
110021	SOL-M™ Blunt Fill Needle 18G*1"	18G	1.2 mm	25 mm	1"
110022	SOL-M™ Blunt Fill Needle 18G*1 1/2"	18G	1.2 mm	40 mm	1 1/2"
110022F	SOL-M™ Blunt Fill Needle w/ Filter 18G*1 1/2"	18G	1.2 mm	40 mm	1 1/2"

Sales Unit: 100 pcs/Box; 1000 pcs/Case

For more information, please contact:

CORP-BF-117-0814

Sol-Millennium Medical Inc.  
315 Shawnee North Drive, Suite 100,  
Suwanee, GA 30024  
T: 404-973-2200  
e-mail: info@sol-m.com

sol-m.com

